

mobileg[™] Ultra Spring Assisted Crutches



Superior Design Combined with New Technology
Say goodbye to traditional crutches with Mobileg, the next generation crutches that deliver the highest level of fit, comfort and performance.



Ultra Safety Saddle

- Flexible mesh distributes weight evenly
- Spring absorbs pressure and motion, minimizing chafing
- Pivots to allow users comfortable, fluid motion
- Saddle rotates, properly aligning shoulders, arms and hands



Ultra Ergonomic Handle

- Neutral hand position distributes weight optimally
- Conforms to hand, distributing pressure and weight evenly for better load-bearing



Adjust and Fit

- One SKU accommodates majority of users--96% of US adult population.
- Height is adjustable from 4'9" to 6'4"
- 300 lbs weight bearing capacity

Ultra Fit Guide

- 1 Locate left and right-handed Mobilegs
- 2 Squeeze push buttons to adjust
- 3
 - Set arm length
 - 1. Hold onto grip
 - 2. Place saddle snug underarm, keeping shoulders relaxed
 - 3. Arm comfortably bent at 15°–30° angle
 - 4. Leave a slight space between armpit and Saddle
 - 5. Adjust to fit
- 4
 - Set leg length
 - 1. Adjust lower section to 6 inches in front of toe when leg is straight in standing position
- 5 Confirm final fit

When necessary, please seek assistance to ensure your safety and comfort during the fitting process.

92% of users find fit & comfort superior to standard crutches*

